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HOFFMANN'S
AQUACC
CARBOXYLATECEMENT

Water-mixable carboxylate cement for lining and luting
single dose. Radiopaque.

In accordance with DIN EN ISO 9917

Made in Germany

10 x 1 g 1 x 1

Instructions for the dentist

Areas of application:

Lining for all filling materials (amalgam, composites, etc.); luting of inlays, onlays, crowns and bridges; retention of orthodontic bands; cementation of endodontic posts and screws; core build-ups; temporary fillings; filling for milk teeth.


Dosage, consistency and timing:

The curing times for the specified conditions were determined in accordance with DIN EN ISO 9917: Mixing at 23 ± 1° C, 50 ± 10% relative humidity and curing at 37 ± 1° C, min. 90% relative humidity. In case of diverging temperature and humidity conditions deviations may occur.

Indication	Consistency	Dosage: Number of drops: 1 stick	Times		
			Mixing time	Working time	Curing time
Luting (for up to 2 crowns)	creamy	5	45 s	2:00 min	3:30 – 4:30 min
Lining	packable	4	45 s	1:30 min	3:00 – 4:00 min


Preparation:

Fill the provided pipette with distilled water. Please note the shelf life of the distilled water used!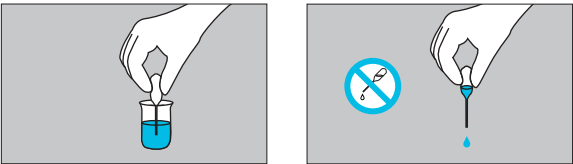

Mixing:

For mixing we recommend Hoffmann's mixing slab (REF 82900) and Hoffmann's spatula (REF 82902). Depending upon the required indication (see table) drip the appropriate water drop quantity on the mixing slab. To do so, hold the pipette vertically and release the desired quantity by applying light pressure (caution – no pumping movement!). Open the AquaCC powder stick pack at the tear seam and completely empty onto the mixing slab. We recommend splitting the powder into three portions (1/2, 1/4, 1/4). Starting with the largest portion, each portion is mixed into the water with a spatula for 15 seconds. Total mixing time of 45 sec. The spatula should be moved in a sweeping fashion – not a stirring movement.

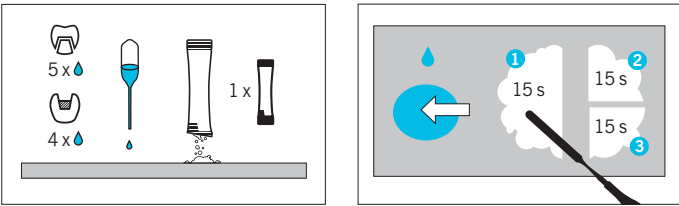Note:

- If necessary, cover the open pulp before use (we recommend Hoffmann's Pulpine, Hoffmann's Pulpine NE or Hoffmann's Pulpine Mineral).
- If necessary, seal the pulp area (we recommend Hoffmann's Copal Varnish).
- The cement should be applied to a preferably dry tooth surface.
- Do not use after the expiry date.

REF 8037010 (universal), 8037014 (light pink)

Date of issue: 06/2026

Hergestellt von/manufactured by

HOFFMANN DENTAL MANUFAKTUR GMBH

Wangenheimstraße 37/39
14193 Berlin, Germany
Tel: +49 30 820099-0
Fax: +49 30 8228145
quality@hoffmann-dental.com
www.hoffmann-dental.com