HOFFMANN'S

HOFFMANN'S

Provisorischer Cement

de

HOFFMANN'S

Provisional Cement

en

PROVISORISCHER CEMENT

Provisional Cement

Made in Germany

For further languages visit our website
www.hoffmann-dental.com

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Instruction for use for the dentist

Composition: powder contains: zinc oxide, aluminium hydroxide; liquid contains: o-phosphoric acid

Areas of Application: Temporary fillings

Special Information:

Do not apply Provisional Cement in case of:
Direct application to dentin near the pulp
or to exposed pulp.

Dispensing Instructions:

1.3 g powder : 1.0 g liquid

Processing Recommendations:

A clean, dry glass mixing slab at 18–20 °C and a nondiscolouring spatula are recommended.

Mixing:

Divide the measured quantity of powder into individual portions in order to achieve a homogenous consistency. Begin with the smallest quantity, quickly mix the powder into the measured liquid, a portion at a time. Discard remaining powder or impure liquids. Dentin areas near the pulp must be protected.

Mixing Time: 45 seconds

Working Time: 1 minute (at 23 °C)

Net Setting Time: 1 minute 45 seconds
3 minutes 45 seconds (at 37 °C).

The times indicated apply from the end of mixing. The cement is to be applied to cavity.

Notes:

- Close containers immediately after removing material.
- The liquid contains phosphoric acid and is corrosive. In the event of accidental eye contact, rinse immediately with large quantities of water.

Package Sizes: Powder: 50 g, liquid: 40 ml

HOFFMANN'S PROVISIONAL CEMENT

- For dental use only
- Do not use after the expiration date
- Material safety data sheet (MSDS) available at www.hoffmann-dental.com

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