HOFFMANN'S ENDO ABSORB

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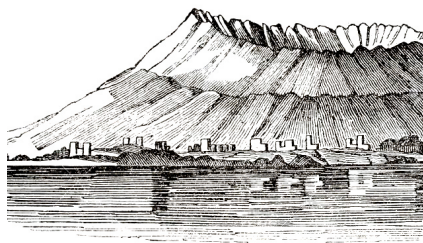
Poudre: 5 g, Liquide: 10 ml

HOFFMANN'S
Dental materials inspired by nature

HOFFMANN'S
ENDO ABSORB

HOFFMANN'S ENDO ABSORB Temporary root canal therapeutic for absorbing apical exudate. Radiopaque

en

HOFFMANN'S ENDO ABSORB

(formerly Proxicalcine B)

Instruction for use for the dentist

Composition:

Powder contains: calcium compounds, bentonite, radiocontrast agent

Liquid contains: alcohol

Areas of application:

Temporary treatment of infected root canals after pulp removal or gangrene treatment. Absorption of apical exudate after root canal preparation. For the maximal duration of one week.

Exact dispensing for optimal material features:

0.24 g powder : 0.15 g liquid

Recommendation for practicable dispensing with enclosed dosing spoon:

3 level spoons of powder : 8 drops of liquid

Shake the glass bottle to loosen the powder before removal. Fill the spoon and use a spatula to level it off. When doing so, do not compress the powder. The dropper of the liquid bottle must be free of any liquid. Hold the pipette in a vertical position.

Mixing:

For mixing we recommend Hoffmann's glass mixing slab (Order No. 82900) and Hoffmann's spatula (Order No. 82908). The powder is mixed into the liquid to form a thin consistency. Any remainders of mixed powder and liquid are to be discarded.

Working recommendations:

Take the thin mixture with a lentulo spiral manually and quickly introduce into the root canal. If necessary, dry with compressed air.

Mixing time: 20 seconds

Setting time: 1:00 – 4:00 minutes

Absorption capacity: 1 week

Special Information:

- To achieve optimum absorption of the exudates, insert Hoffmann's ENDO ABSORB into dry root canals.
- An x-ray to check the root canal filling is always recommended.
- We recommend Hoffmann's ENDO ABSORB PLUS for a longer application time.
- Do not use after the expiration date.

Safety instructions:

- Keep containers closed and always close immediately after use.
- The liquid contains alcohol. Keep away from open flames and sources of ignition.
- Safety data sheets available at www.hoffmann-dental.com.

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