HOFFMANN'S

GUTTAPERCHA

Made in Germany

For further languages visit our website
www.hoffmann-dental.com

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HOFFMANN'S

Guttapercha

Instruction for use for the dentist

Composition:

zinc oxide, gutta-percha, paraffin, beeswax

Area of Application:

For temporary sealing of cavities not exposed to strong chewing forces and not containing fluid medicines. For temporary sealing for holes in implant screws.

Special information:

Do not apply GUTTAPERCHA in case of:
Sealing of insertions, applications on opened dental pulp, cavities containing liquide medicines.

Dispensing:

Cut the required quantity from the GUTTAPERCHA stick.

Processing: heat the GUTTAPERCHA to 60 – 65 °C on a hot plate. the heating may also be performed using a small flame. avoid overheating and direct contact to the flame, as this causes its decomposition. Knead and apply the heated material with a metal spatula.

Package Sizes: 110 g

HOFFMANN'S GUTTAPERCHA

- For dental use only
- Do not apply when too hot, otherwise thermal damage to pulp/dentine
- a suitable pulp/dentine protection is necessary for vital teeth
- Store in closed containers and protect from light
- Do not use after the expiration date