O₃

HOFFMANN'S PeriO₃ Oil

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Conditionnement:
3 x 1 ml

HOFFMANN'S
Dental materials inspired by nature

HOFFMANN'S
PeriO₃ Oil

HOFFMANN'S PERIO₃ OIL

Organic gum therapy

en

HOFFMANN'S PeriO₃ Oil

(formerly Proxingivine)

Instructions for dental care professionals

Composition:

Ozone-enriched olive and castor oil.

Areas of application:

Treatment of chronic and acute gingivitis, support of periodontitis treatment, application in case of gum traumas after crown preparation. Inhibition of plaque formation through physical wetting of the teeth and gums.

Dosage:

The dosage depends on the application: Use Hoffmann's PeriO₃Oil selectively in cases of chronic and acute inflammation of the gums and gum traumas after crown preparation, apply in gingival pockets to support the treatment of periodontitis.

Working recommendations:

Prior to treatment, a thorough oral cleaning is recommended (if necessary a professional tooth cleaning (PTC), depending on the indication a root debridement). Screw the supplied blunt cannula onto the syringe firmly and, if necessary, bend for the treatment of hard-to-reach areas. Apply Hoffmann's PeriO₃Oil into the interdental spaces or the periodontal pockets using the cannula. Creating a sealing gum adhesion bond, e.g. by means of prosthesis adhesive, extends the adhesion on teeth and gums and as a result the treatment time. Four applications in a two-day rhythm are recommended. Ideally patients should not eat any solid food in the next 12 hours. Therefore application in the evening hours is recommended. After the treatment, dental inspection is necessary.

Effect duration:

Up to 48 hours

Note:

- Stable up to the expiry date if stored in a refrigerator between 6 and 8 °C.
- Shelf life of 4 weeks if stored at room temperature between 20 and 25 °C.
- Protect from direct sunlight.
- Hoffmann's PeriO₃Oil cannulas are for single use.
- The simultaneous application of mouthwash (e.g. chlorhexidine) or other gum therapeutic agents may have an adverse effect on the efficacy of the product.
- During pregnancy the daily dose of 4 ml must not be exceeded.
- For tips and application please find an information video on our website: <https://hoffmann-dental.com/>

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