HOFFMANN'S PROXI APEX

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Poudre: 10 g, Liquide: 10 ml

HOFFMANN'S
Dental materials inspired by nature

HOFFMANN'S
PROXI APEX

HOFFMANN'S PROXI APEX Calcium hydroxide-containing
root canal filling material based on zinc oxide eugenol. Radiopaque

en

HOFFMANN'S PROXI APEX

(formerly Proxiapex)

Instruction for use for the dentist

Composition:

Powder contains: calcium compounds, zinc compounds, zirconium oxide
Liquid contains: eugenol

Areas of application:

Permanent filling of root canal after vital extirpation, gangrene treatment or before apicoectomy

Exact dispensing for optimal material features:
0.23 g powder : 0.10 g liquid

Recommendation for practicable dispensing with enclosed dosing spoon:
2 level spoons of powder: 2 drops of liquid

Shake the glass bottle to loosen the powder before removal. Fill the spoon and use a spatula to level it off. When doing so, do not compress the powder. The dropper of the liquid bottle must be free of any liquid.

Mixing:

For mixing we recommend Hoffmann's spatula (Order No. 82908). The powder is mixed into the liquid to form a creamy consistency. Any remainders of mixed powder and liquid are to be discarded.

Working recommendations:

After drying of the root canal take the creamy mixture with a lentulo spiral and manually introduce into the root canal. For the seal with a gutta-percha tip it is recommended to cover it with the spatula with the creamy mixture of Hoffmann's PROXI APEX and then to slowly insert it into the canal.

Mixing time: 30 seconds
Setting time: 4:00 – 15:00 minutes

Special Information:

- Introduce Hoffmann's PROXI APEX only in dry root canal.
- An x-ray to check the root canal filling is always recommended.
- In combination with a filling, a lining with Hoffmann's Phosphate Cement is recommended.
- Do not use after the expiration date.

Information for people with allergies:

Do not use Hoffmann's PROXI APEX in case of a known eugenol allergy.

Safety instructions:

- Keep containers closed and always close immediately after use.
- Safety data sheets available at www.hoffmann-dental.com.

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