# HOFFMANN'S PULPINE

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Poudre: 10 g, Liquide: 10 ml

**HOFFMANN'S**  
*Dental materials inspired by nature*

**HOFFMANN'S**  
**PULPINE**

## HOFFMANN'S PULPINE

Eugenol-containing pulp capping material

en

## HOFFMANN'S PULPINE

(formerly Proxipulpine Standard)

### Instruction for use for the dentist

#### Composition:

Powder contains: calcium compounds, zinc compounds

Liquid contains: ethanol, propolis, eugenol

#### Areas of application:

For direct and indirect pulp capping after full or partial caries removal in cases of initial pulpitis, in cases of caries profunda and iatrogenic injury of dental pulp.

#### Exact dispensing for optimal material features:

0.14 g powder : 0.10 g liquid

#### Recommendation for practicable dispensing with enclosed dosing spoon:

1 level spoon of powder : 2 to 3 drops of liquid

Shake the glass bottle to loosen the powder before removal. Fill the spoon and use a spatula to level it off. When doing so, do not compress the powder. The dropper of the liquid bottle must be free of any liquid.

#### Mixing:

For mixing we recommend Hoffmann's spatula (Order No. 82908). The powder is mixed into the liquid to form a thin (non-dripping) consistency. Any remainders of mixed powder and liquid are to be discarded.

#### Working recommendations:

Take up the thin (non-dripping) mixture with a ball-shaped instrument - such as a ball plugger or a blunt probe - and coat the affected area thinly.

Mixing time: 20 seconds

Setting time: 1:00 - 3:45 minutes

#### Special Information:

- In the case of one or more symptoms, pulpitis pain, already necrotic pulp with or without clear pain to heat or non-response to cold test, the success rates of the treatment with Hoffmann's PULPINE decrease.
- The iatrogenic, accidental and restricted pulp opening should not exceed 1 mm<sup>2</sup>.
- In combination with a filling, a lining with Hoffmann's Phosphate Cement is recommended.
- After 1 or 2 weeks a dental follow-up is required.
- Do not use after the expiration date.

#### Information for people with allergies:

Do not use Hoffmann's PULPINE in case of a known Eugenol or bee products allergy.

#### Safety instructions:

- Keep containers closed and always close immediately after use.
- The liquid contains ethanol. Keep away from open flames and sources of ignition.
- Safety data sheets available at [www.hoffmann-dental.com](http://www.hoffmann-dental.com).

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