HOFFMANN'S PULPINE NE

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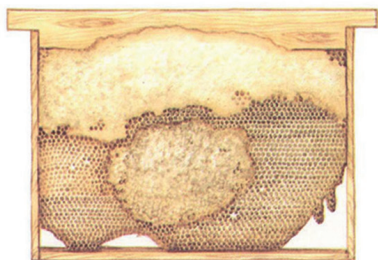
Flammable liquid

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Poudre: 10 g, Liquide: 10 ml

HOFFMANN'S
Dental materials inspired by nature

HOFFMANN'S
PULPINE NE

HOFFMANN'S PULPINE NE

Eugenol-free pulp capping material

en

HOFFMANN'S PULPINE NE

(formerly Proxipulpine 1,9 NE)

Instruction for use for the dentist

Composition:

Powder contains: calcium compounds, zinc compounds

Liquid contains: ethanol, propolis

Areas of application:

For direct and indirect pulp capping after full or partial caries removal in cases of initial pulpitis, in cases of caries profunda and iatrogenic injury of dental pulp.

Exact dispensing for optimal material features

0.19 g powder : 0.10 g liquid

Recommendation for practicable dispensing with enclosed dosing spoon

1 level spoon of powder : 3 to 4 drops of liquid

Shake the glass bottle to loosen the powder before removal. Fill the spoon and use a spatula to level it off. When doing so, do not compress the powder. The dropper of the liquid bottle must be free of any liquid.

Mixing:

For mixing we recommend Hoffmann's spatula (Order No. 82908). The powder is mixed into the liquid to form a thin (non-dripping) consistency. Any remainders of mixed powder and liquid are to be discarded.

Working recommendations:

Take up the thin (non-dripping) mixture with a ball-shaped instrument - such as a ball plugger or a blunt probe - and coat the affected area thinly.

Mixing time: 20 seconds

Setting time: 1:00 – 3:45 minutes

Special Information:

- In the case of one or more symptoms, pulpitis pain, already necrotic pulp with or without clear pain to heat or non-response to cold test, the success rates of the treatment with Hoffmann's PULPINE NE decrease.
- The iatrogenic, accidental and restricted pulp opening should not exceed 1 mm².
- In combination with a filling, a lining with Hoffmann's Phosphate Cement is recommended.
- After 1 or 2 weeks a dental follow-up is required.
- Do not use after the expiration date.

Information for people with allergies:

Do not use Hoffmann's PULPINE NE in the case of a known bee products allergy.

Safety instructions:

- Keep containers closed and always close immediately after use.
- The liquid contains ethanol. Keep away from open flames and sources of ignition.
- Safety data sheets available at www.hoffmann-dental.com.

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