HOFFMANN'S READY2MIX

PHOSPHATECEMENT NORMAL

REF 80024-(*Colour-No.)
MADE IN GERMANY

CE 0482

HOFFMANN'S READY2MIX
NORMALHÄRTEND

de

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HOFFMANN'S READY2MIX
NORMAL SETTING

The logo consists of the letters 'en' in a lowercase, sans-serif font, enclosed within a thin circular border.

Zinc phosphate cement, x-ray visible in single doses
for cementation and cavity lining
In accordance with **DIN EN ISO 9917**

INSTRUCTION FOR USE FOR THE DENTIST

COMPOSITION

Powder contains: zinc oxide, magnesium oxide;
Liquid contains: o-phosphoric acid.

AREAS OF APPLICATION

Cavity lining as basis for all filling materials (amalgam, composites etc.), cementation of inlays, crowns and bridges (also zirconia ceramics) and crowns and bridges on implants, cementation of orthodontic bands, retention pins and screws, cement build up, temporary fillings.

DISPENSING INSTRUCTIONS

FOR CEMENTATION: (enough material up to 3 crowns)

7 drops of liquid + 1 powder stickpack

FOR CAVITY LINING:

5 drops of liquid + 1 powder stickpack

MIXING

You will require a clean and dry glass slab. We recommend Hoffmann's mixing block (# 82900) and Hoffmann's spatula (# 82902). Apply the required amount of drops of READY2MIX liquid onto the glass slab. To do so hold the tube in an upright position. Build up pressure with index finger and thumb and keep upright up to the last drop (attention: no pumping action). Open READY2MIX powder stickpack at the perforation and empty completely onto a clean and dry glass slab. For best possible mixing results, we recommend that the powder be divided into single portions ($\frac{1}{4}$, $\frac{1}{4}$, $\frac{1}{2}$). The entire powder is mixed into the liquid with a spatula as follows: **2 x $\frac{1}{4}$ in 15 seconds each and $\frac{1}{2}$ in 30 seconds.** The spatula should be moved in a sweeping fashion – not a stirring movement.

NOTE: View an instructional film on mixing at www.READY2MIX.de

TIMES

MIXING TIME: 60 seconds

PROCESSING TIME from end of mixing (at 23 °C/ 72°F):

FOR CEMENTATION **3:00** minutes

FOR CAVITY LININGS **1:30** minutes

SETTING TIME from end of mixing (at 23 °C/ 72°F):

FOR CEMENTATION **5:00 – 7:30** minutes

FOR CAVITY LININGS **3:30 – 5:30** minutes

SPECIAL INFORMATION

- Do not shake the tube before extracting liquid
- It is vital that dentin near the pulp or opened pulp is covered with a calcium hydroxide compound.
- After inserting the restoration, short-term irritation of the pulp could occur (acid), however, this irritation will recede completely after a few minutes.
- The cement should be applied to a preferably dry tooth surface
- The liquid contains phosphoric acid and is corrosive. In the event of accidental eye contact, rinse immediately with large quantities of water.

- Do not use after the expiration date

* COLOURS/SHADES

-01 (white), -02 (bluish white), -03 (yellowish white),
-04 (light yellow), -05 (yellow), -06 (gold yellow),
-07 (gold brown), -08 (pearl grey), -09 (grey),
-10 (greenish grey), -11 (bluish grey), -12 (brown),
-13 (greyish brown), -14 (light pink), -15 (pink)

READY2MIX is also available as Hoffmann's Cement quick setting and Hoffmann's Carboxylate Cement.

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Hoffmann Dental Manufaktur GmbH