HOFFMANN'S

SILAN

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HOFFMANN'S SILAN

en

INSTRUCTION FOR USE FOR THE DENTIST

Description of the product

Hoffmann's SILAN is a two-component silanizing agent. It is
to be used for silanization of the surfaces of full ceramic resto-
rations to establish a bond to any fixation material containing
methacrylate groups.

Composition

Component A contains: acetic acid in a mixture of ethanol and
water; component B contains: 3-methacryloxypropyl-trimethoxy-
silane in a mixture of ethanol and water.

Areas of application

Surfaces of full ceramic restoration (bridges, crowns, inlays,
Onlays, Veneers) are silanized by Hoffmann's Silan to establish
a bond to fixation composites, -compomers or -ormocers contain-
ing methacrylate groups within the adhesive technology.

Contraindications

Hoffmann's SILAN is not to be used in case of a known allergy
to one of the ingredients.

Instructions for use

Preparation

1. After test-placing of the full ceramic restoration and possible
corrections, the parts to be fixed are etched by any hydrofluoric
acid gel. Thereby the manufacturer's instructions have to be
observed strictly. The areas etched and cleaned by the hydro-
fluoric acid gel are to be dried well with an oil-free air stream.
Afterwards avoid their contamination with humidity, saliva and
others in any case.
2. The tooth must be prepared according to the rules of the
adhesive technology.

Use of Hoffmann's Silan

Component A and component B should be mixed at a ratio
of 1:1 immediately before application. Put the same number
of drops of each component into a mixing flask. Immediately
close the flask after use again. Stir the mixture with a plastic or
metallic instrument and apply them thinly with a brush

foamed material- or cotton pellet on the etched surfaces of
the full ceramic restoration. After a residence time of about
30 seconds, remove the excess mixture with an air syringe
and dry the surface.

Attention

After the drying, the surface still looks slightly glossy. Do
not wipe! HOFFMANN'S SILANE is durable in the mixed
state for about

The product is only for the dental use. It must be applied
according to the instructions for use. The manufacturer does
not accept liability for any damage arising out of another or
inappropriate application.

Side effects

Are not known.

Warning notices

- Hoffmann's SILAN contains a silane, ethanol and acetic
acid. That's why use caution to prevent the product from
coming in contact with the skin and mucosa or getting into
the eye. In case of an accidentally contact with the human
body, wash off with copious amounts of water. If the pro-
duct gets in the eye, immediately wash the eye with
copious amounts of water and consult an eye specialist.
- Keep medical devices out of reach of children!
- Do not use after the expiration date.

Storage precautions

Storage at 2-28 °C. Keep away from heat and sunlight.

Package sizes

Package with 5 ml component A and 5 ml component B.

Safety precautions

F – The product is highly inflammable.

The product contains ethanol.

Keep away from unshielded flame and ignition source.

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Coupling Agent

for the surfaces of all-ceramic restorations

www.elfu.hoffmann-dental.com

REF 82101

Hergestellt von/manufactured by
HOFFMANN DENTAL MANUFATUR GMBH

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