|                                                                         |               |
|-------------------------------------------------------------------------|---------------|
| <div>HOFFMANN'S CEMENT<br/>schnellhärtend<br/>Zinkphosphat-Cement</div> | <div>de</div> |
|-------------------------------------------------------------------------|---------------|

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**Instruction for use for the dentist**

**Composition:**  
Powder contains: zinc oxide, magnesium oxide; Liquid contains: o-phosphoric acid.

**Areas of Application:**  
Cavity lining as basis for all filling materials (amalgam, composites etc.), cementation of inlays, onlays, crowns and bridges made of precious metal, non-precious metal as well as metal ceramics (zircon, aluminium and lithium disilicate ceramics) and crowns and bridges on implants, cementation of orthodontic bands, retention pins and screws, cement build up, temporary fillings.

**Dispensing Instructions:**  
Cementation consistency: 1.8g powder : 1.0g liquid  
Cavity lining consistency: 2.25g powder : 1.0g liquid  
Dispensing and testing climate for testing in accordance with DIN EN ISO 9917: Mix quantities indicated at 23±1°C and a relative humidity of 50±10%, he dispensing instructions also apply when mixing the liquid with HOFFMANN´S UNIVERSAL CEMENT (REF 80032-).

**Mixing:**  
A clean, dry glass plate at ca. 20°C (68°F) and a non-discolouring spatula should be used for mixing. We recommend Hoffmann´s glass mixing slab (REF 82900) and Hoffmann´s spatula (REF 82902). In order to achieve a homogenous consistency, divide the measured quantity of powder into 4 portions (½ ¼ ¼ ¼). Beginning with the smallest quantity, stroke the powder into the measured liquid within 90 seconds. The cavity lining consistency has been reached when on lifting the spatula, the peak drawn out forms a small hook and does not sink back into the paste (Fig. 1). The cementation consistency has been reached when the peak formed on lifting the spatula slowly falls back into the paste (Fig. 2 and 3). Discard remaining powder or impure liquid.

**Mixing Time:** 90 s  
**Processing time from beginning of mixing (at 23°C, 72°F):**  
For cementation: 2:30 min  
For cavity linings: 2:00 min  
**Setting Time from end of mixing:**  
For cementation: 3:00 — 5:30 min  
For cavity linings: 2:30 — 4:00 min

- Special Information:**
- It is vital that opened pulp is covered. According to the indication we recommend Hoffmann´s PULPINE (REF 840-16/17), Hoffmann´s PULPINE NE (REF 840-56/57), Hoffmann´s PULPINE MINERAL (REF 840-78/77) or Hoffmann´s ENDO REPAIR (REF 840-86/87) as capping material.
  - By deep cavity short-term irritation of the pulp could occur, however, this irritation will recede completely after a few minutes. For cavity lining we recommend to use Hoffmann´s Copal Varnish (REF 81901).
  - To obtain an optimal pH-value, comply with the mixing time of 90 s.
  - The cement should be applied to a preferably dry tooth surface.
  - The liquid contains phosphoric acid and is corrosive. In the event of accidental eye contact, rinse immediately with large quantities of water.
  - Close containers immediately after removing material.

**Package Sizes:**  
 powder 100g |  liquid 40ml |  30g powder + 15ml liquid

**HOFFMANN´S CEMENT normal setting**  
• Do not use after the  expiration date.  
• Compliant with DIN EN ISO 9917

|                                                                                     |                        |                  |                     |
|-------------------------------------------------------------------------------------|------------------------|------------------|---------------------|
|  | <b>Colours/Shades:</b> |                  |                     |
|                                                                                     | -01 white              | -02 bluish white | -03 yellowish white |
|                                                                                     | -04 light yellow       | -05 yellow       | -06 gold yellow     |
|                                                                                     | -07 gold brown         | -08 pearl grey   | -09 grey            |
|                                                                                     | -10 verdosa grey       | -11 bluish grey  | -12 brown           |
|                                                                                     | -13 greyish brown      | -14 light pink   | -15 pink            |

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